- Insulated stainless steel cabinet
- Frost resistant down to (-40 °C)
- Additional frontal heating element (-20 °C to -40 °C)
- Fan for ventilation
- 2 x LED lights (meaning defibrillator is visible in the dark)
- Designed to be easy to open, even under conditions of heavy snow and frost.
- Future-proof! (Software can be upgraded)

---

**FUNCTIONAL DESCRIPTION:**

- If the door of the cabinet is opened... The SIREN immediately sounds.  
(if the door of the cabinet is closed once again within 3 minutes the siren will stop automatically)
- Once the door is closed... The cabinet is once again ready for operation.

---

**TECHNICAL DESCRIPTION:**

- Door: The door is fitted with thermopane glass and marked with international AED labels/symbols.  
 Light: 2 x LED lights in the cabinet, such that the defibrillator is visible in the dark.  
 Siren: Siren in the cabinet. (100 dBA / 1m)  
 Fan: Ventilation fan, starts automatically when temperature exceeds +40 °C.  
 Frost-proof cabinet: Thermostatically-controlled heating element. Frost resistant down to -40 °C  
 Voltage: Must be connected to a 230V (116Wh) electrical supply. Outdoor cable D8 mm (3 x 1.5 mm<sup>2</sup>)
- External dimensions: W x H x D = 44 x 46 x 27 cm (incl. door handles)  
 Internal dimensions: W x H x D = 33 x 38 x 20 cm
- Weight: Gross weight incl. packaging 18.5 kg.

---

**MAINTENANCE / CLEANING:**

We recommend cleaning the outer surface of the cabinet at least 1-2 times per year.  
 Wipe using detergent and a damp cloth or brush and rinse with clean water. Finally wipe down the cabinet with a dry cloth.

If the surface of the cabinet becomes tarnished, this will be due to deposits which can easily be cleaned off.  
 For difficult deposits, use a scouring pad. **Never use wire wool.**

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